

Firmware Update Instructions


Step 1:

Remove the battery door and battery as shown. If your unit has a **Red Switch**, then move it to the RIGHT position for firmware update function.




Step 2:

Depending on the E3 version you have connect either a Mini-B USB cable (digital camera type cable) or Micro USB cable (android phone type cable) to the E3 controller end and connect the standard USB end to your laptop or PC as shown.

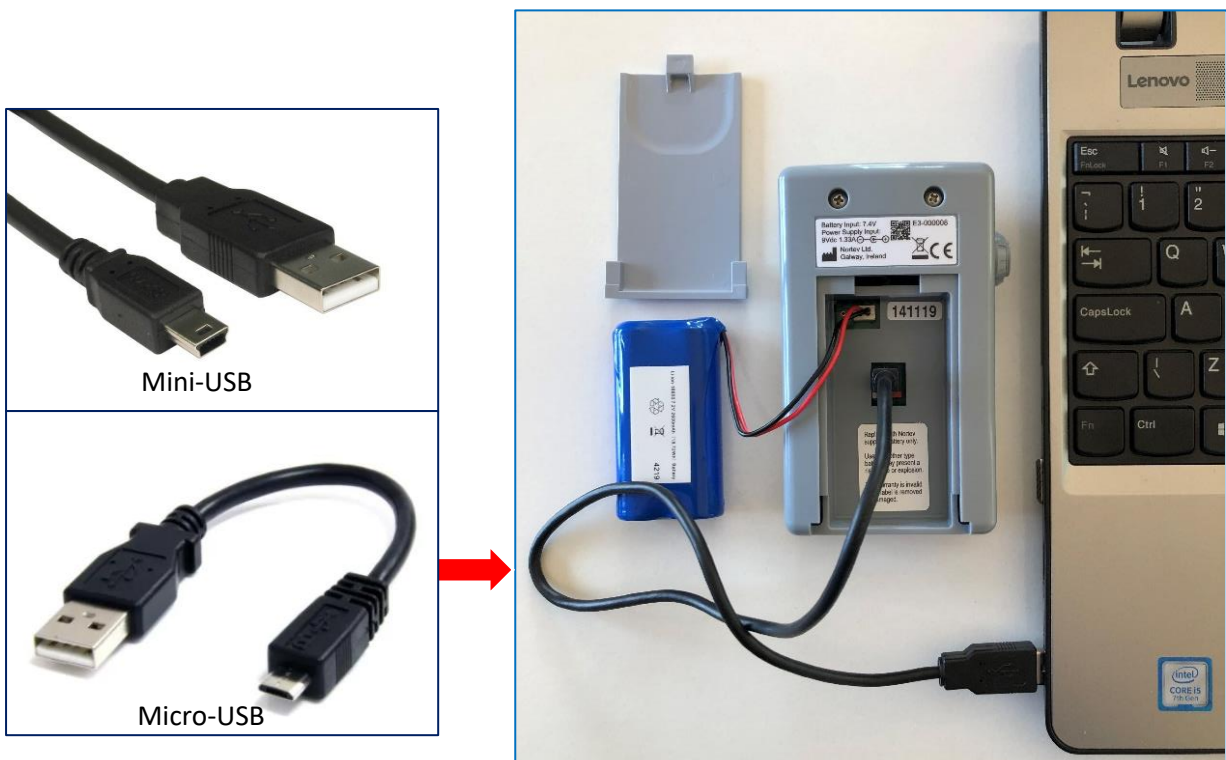

Step 3:

Go to www.flexineb.com, and click on the **Information** tab & then **Firmware Update**

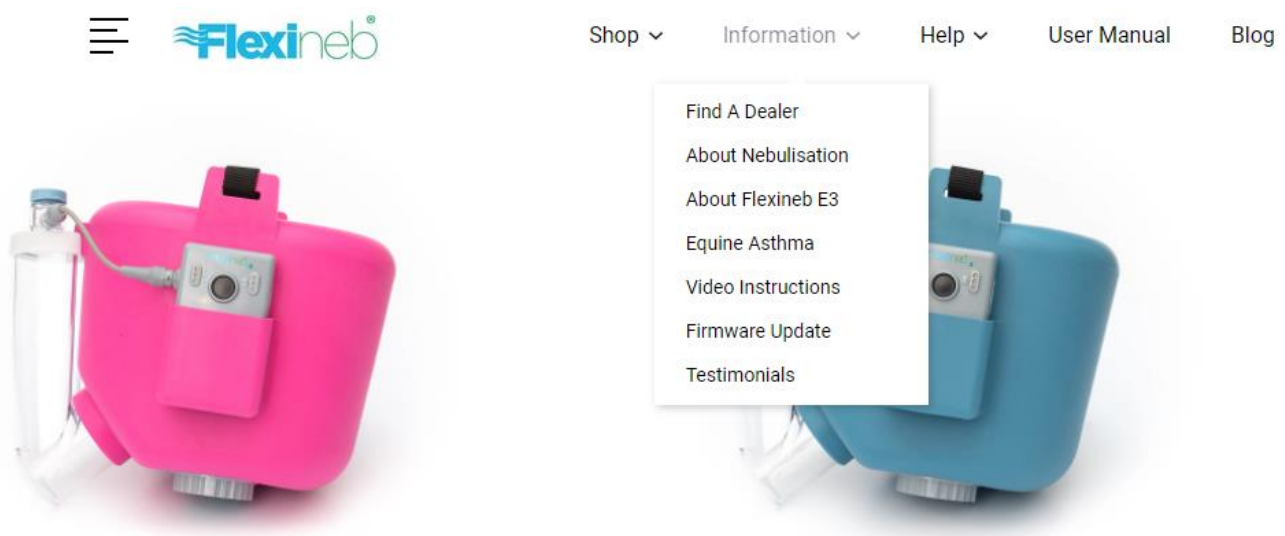

Step 4:



Firmware Update

Click on Download for Windows



Flexineb® E3 Firmware Application

[Download Instructions \(PDF\)](#)

[Download for Windows](#)

Release Notes

Date Modified: 28.06.2021

- Micro-USB replaces Mini-USB connection on PCB for firmware updates.
- Red Boot switch removed.

Step 5:

Firmware Update

Flexineb® E3 Firmware Application

[Download Instructions \(PDF\)](#)

[Download for Windows](#)

Release Notes
Date Modified: 28.06.2021

- Micro-USB replaces Mini-USB connection on PCB for firmware updates.
- Red Boot switch removed.

Click on the downloaded FlexinebE3 Firmware.exe file

Westlink Commercial Park,
Co. Galway,
Ireland.
(091) 799 778

Quick Links

- Information
- Help
- User Manual
- Blog

Categories

- Flexineb Systems
- Spare Parts
- [View All](#)

Flexineb-E3-Firmw...exe

Step 6:

Windows protected your PC

Microsoft Defender SmartScreen prevented an unrecognized app from starting. Running this app might put your PC at risk.

[More info](#)

Click on More Info

Don't run

Step 7:

Windows protected your PC

Microsoft Defender SmartScreen prevented an unrecognized app from starting. Running this app might put your PC at risk.

Application: Flexineb-E3-Firmware-Update (1).exe
Publisher: Unknown publisher


Click on Run anyway

Run anyway Don't run

Step 8:

The program will search for your unit before proceeding to the **Update** function


Firmware Update

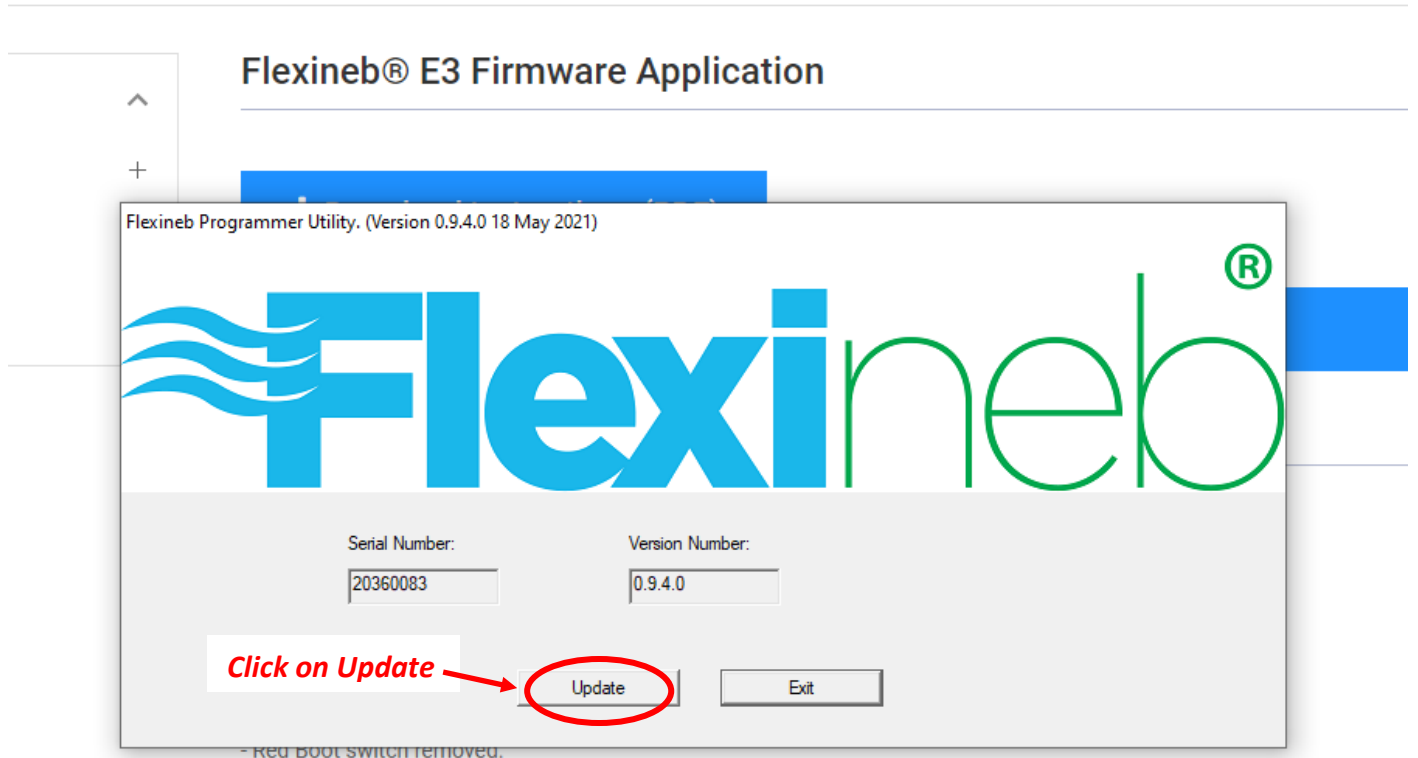

Step 9:

[Home](#) > [Information](#) > [Firmware Update](#)

Firmware Update

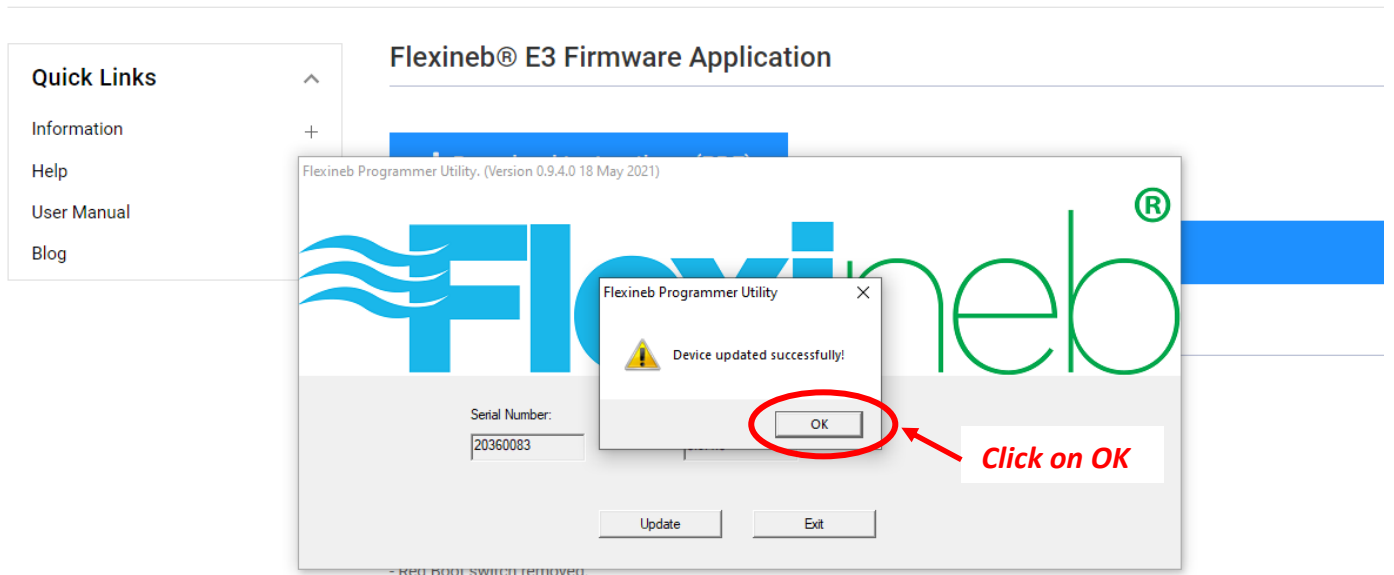

Step 10:

Home > Information > Firmware Update

Firmware Update

Quick Links

- Information
- Help
- User Manual
- Blog

Flexineb® E3 Firmware Application

Flexineb Programmer Utility. (Version 0.9.4.0 18 May 2021)


Serial Number: 20360083 Version Number: 0.9.4.0

Update **Exit** *Click on Exit*

- Red BOOT SWITCH removed.

Step 11:

Move **Red Switch** back to the LEFT position for normal function on units with this switch.
Re-fit the battery and add the battery door. All done.

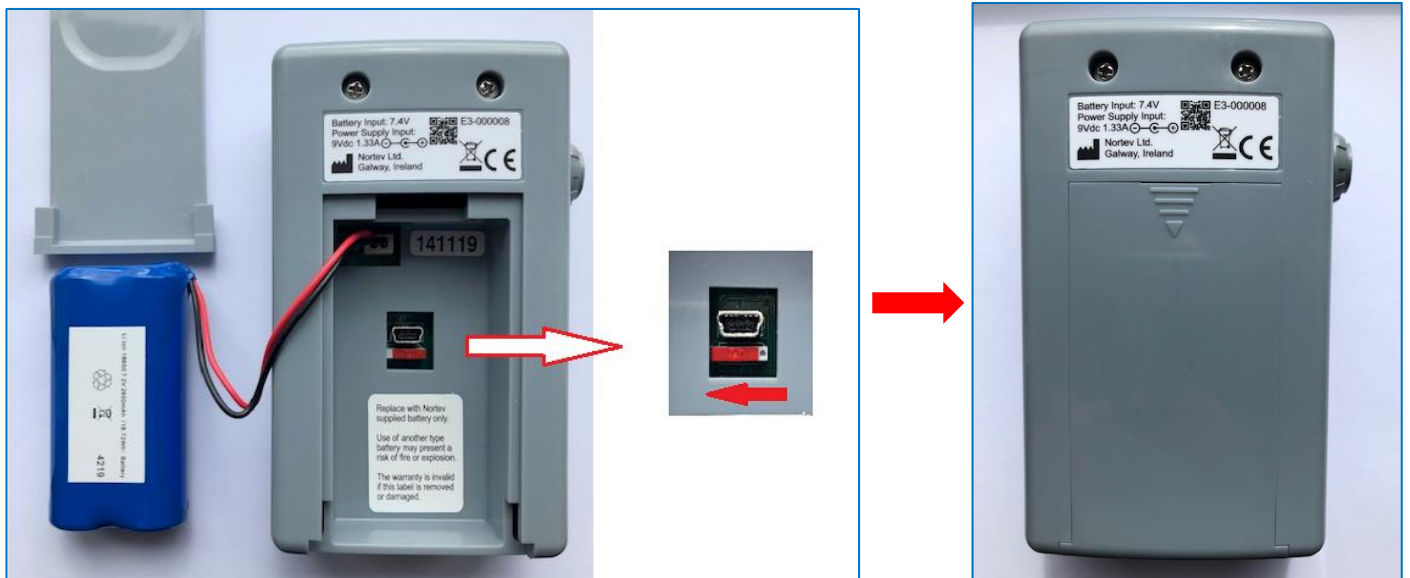

END

